Pekalux POWER LED

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Pekalux POWER LED

User Manual

1. Contents
2. Introduction
3. Unpacking & Installation Guide
4. Safety Precautions
5. How to use Pekalux POWER LED
6. Maintenance & Storage Tips
7. Troubleshooting
8. General Information
9. Warranty & Service

2. Introduction

Thank you for purchasing Pekalux POWER LED of Kulzer GmbH.

Notice: Before using this product, please read this manual carefully, and keep it in a safe place for future reference.

2.1 Main Features

- High light power
- Excellent readability through the customized LED display
- Multifarious modes for convenient access
- Use interchangeable battery pack
- Built-in power intensity meter
- Wireless system (use lithium-Ion battery)
- Additional available wired power addaptor (DC Jack) for continous wired operation

2.1 Parts of Pekalux POWER LED

- 1 Charger stand
- 2 Battery pack
- 3 Handpiece
- 4 Anti- glare shield
- 5 Light guide
- 6 Power adapter
- 7 DC Jack wired power adapter

3. Unpacking & Installation Guide

3.1 Unpacking

- Take all parts of product out of the box.
- Check that there are no broken or missing parts.
- Preparing future upgrades and repairs, keep the original box.
- Get rid of packing materials in accordance with the related regulations.

3.2 Installation Guide

2

- 1 Connect the plug of power adapter to charger stand, and set the stand on a flat surface. When installed properly, the green power indicator will be on.
- 2 Remove the safety film of the battery contacts.
- 3 Put the battery pack to handpiece paying attention to the direction. Alternatively you can use the DC Jack power adapter if you prefer working with a wired device. Therefore insert the DC Jack into the handpiece and connect with the power cord.
- 4 Put to slide the light guide to the handpiece tightly, or it might be fall out, or can be a cause of lower output of the power.
- 5 Put the anti-glare shield from the point of light guide and set it on a proper position.

4. Safety Precautions

4.1 General Safety Precautions

- Before using this instrument, please read its manual carefully.
- Please use our original genuine parts only, or it can be a cause of damage to the product, and also, the product could not receive warranty service.
- Please keep terminals of the battery clean with a dry cotton swab, etc.
- Please disconnect the plug of charger when not in use for long time.
- When not in use for long time, disjoint battery and handpiece separately and keep them in a safety and clean place.
- Never carry out the ultrasonic, boiling, dry heat, Steam or cresol disinfection. It might be a serious cause of trouble.

- Do not leave this instrument in the place where is high temperature or humidity.
- Do not subject the instrument to severe impact or drop it from high position. This might be the cause of trouble.
- Be careful of the charger cable do not be weighted down or chopped by other heavy objects.
- Do not get rid of the batteries close to the fire. It may be a cause of explosion or big fire.
- Get rid of the batteries at the named place in accordance with the related regulations and the local environmental guidelines.
- Never disassemble, repair or modify this instrument except manufacturer's technicians.

4.2 Precautions for use

- Do not use this instrument with other purpose. It might be a cause of the safety accident.
- Avoid looking directly at the light source of this instrument. It may harm your eyes and vision.
- Do not use a broken and damaged instrument.
- When disinfecting or cleaning, always disconnect charger plug from the power consent.


5. How to use Pekalux POWER LED

5.1 Electric Charging

- When the power connects to charger stand, the green power indicator will be on.
- Upon putting handpiece on the charge stand, the red indicator will be on and start to charge. (If your battery of handpiece is fully charged, the red indicator will not be on.)
- When charging is finished, the red indicator will be off.

5.2 User Interface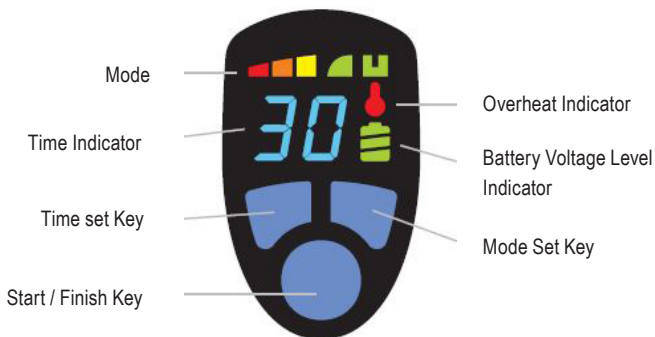

5.3 How to use Pekalux POWER LED


5.3.1 Start and Usage

- By combining the battery, the display window will be on, that shows the instrument is available to be used then.
- Select a desired mode using mode set key, and set a desired irradiation time using time set key.
- Pushing start key, rays of light will be irradiated according to the selected mode and time, and during its operation, the irradiation will be stopped by pushing finish key.
- If it is not used some minutes, (about 5 minutes) it will be diverted its mode to sleep-mode automatically, therefore its battery discharge will be minimized with this standby mode.
- With pushing start key in sleep mode, it will be turned to previous mode.

5.3.2 Mode Description of Pekalux POWER LED


① Middle power Mode

1. Optical output – Over 1800 mW/cm^2 (+/- 10 %)
2. Available time setting – 5, 10, 20, 30 seconds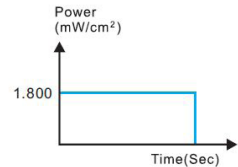


② High power Mode

1. Optical output – Over 3000 mW/cm^2 (+/- 10 %)
2. Available time setting – 5, 10, 15, 20 seconds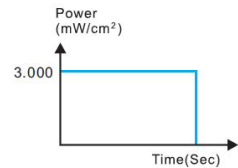


③ Soft Start Mode

1. Optical output – After gradually increasing from 0 to 1800 mW/cm^2 (+/- 10 %) and be maintained until finish time
2. Available time setting – 10, 15, 20 seconds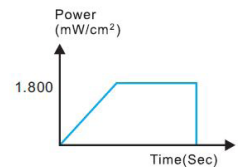

④ Flickering Mode

1. Optical output – 1800 mW/cm^2 (+/- 10 %) Repeat it 10 times at intervals of 1 second.
2. Available time setting – 20, 40, 60, 80 seconds

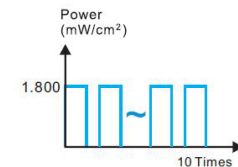! A high power visible light radiation from this instrument can cause eyesight damage. A person using it should wear safety glasses or use anti glare shield.

5.3.2 Status Indication and Warning

1 Battery Level Indication

1. If battery level indicator is all green, it shows fully charged battery, and all change to the red, it shows the time to charge.
2. If you use it without charging after battery level indicator changed to red all, there will be „bA“ on the display and operating will be stopped.

2 Temperature Status Indication

1. If the temperature of the power LED climbs up above the setting temperature, overheat warning indicator will be on.
2. If you use it continuously after overheat indicator is turned on, there will be „oH“ on the display and operating will be stopped until the temperature turn back to normal.

5.3.4 Optical Output Checking

- Over 600 mW/cm² (+/- 10 %)
- Over 1200 mW/cm² (+/- 10 %)
- Over 1800 mW/cm² (+/- 10 %)
- Over 2400 mW/cm² (+/- 10 %)
- Over 3000 mW/cm² (+/- 10 %)

Put the end of the light guide at the middle of the oval shaped indicator and irradiate light. You can check and see the optical output instantly. Please note there can be occurred an accidental error depends on the location and angle of the light guide.

6. Maintenance & Storage Tips

6.1 Maintenance Tips

- Disconnect the connection cable from electrical outlet before cleaning.
- When the light guide become dirty, remove the stain with soft materials such as wood or plastic etc, but a sharp thing like metal.
- Wipe clean the surface of instrument with soft cloth or tissue.
- Use a cotton swab etc. to wipe clean the charging terminal.
- Be careful not to let liquid of sprayer or cleaner into the instrument.

6.2 Storage Tips

- Keep it away from water.
- Keep it away from the places where can be affected from abnormal atmospheric pressure, temperature or humidity, including sunlight, dust, salt, sulfur etc.
- Keep away from the places where any chemicals or gases are.
- Keep away from fire.

- Do not use the ethanol and phenol binder for disinfection.
- Do not take autoclave or thermal disinfection for handpiece or cable.
- Do not immerse the handpiece in a liquid or a fluid.
- No modification of this equipment is allowed.

7. Troubleshooting

- ❶ Battery can not be recharged instead of long time
→ Check the green light of the charger is on first, and if it is off, make sure the power plug and adapters are well connected. When the green light is on, check the battery terminals becomes dirty or if the safety film is still on the contacts of the battery. If so remove the safety film.
- ❷ Handpiece operates anything unusually
→ Check the wrongly pressed keys on the keypad and take out battery and remount, and all the circuit will be reset.
- ❸ No signals on the display of handpiece and not operating
→ Take out battery and and check if the safety film is still on the contacts of the battery. If so remove the safety film, remount and recharge the battery.
- ❹ Output is too low unusually
→ Check the close adhesion of light guide and wipe out the dirt of it.
- ❺ Battery discharge too fast
→ Rechargeable lithium-ion battery for this product has its own service life of charge cycles. If you find a problem on charging, replace it to new one.

If you cannot solve the problems despite the above measures, please contact the dealer or Heraeus Kulzer GmbH, they will show you the way to solve kindly.

8. General Information

8.1 Specifications

- Light source: Power LED
- Wave length range: 430 ~ 490 nm
- Light power: Max 3.000mW²/cm (±10 %)
- Battery: 3,7 V Lithium-Ionen (LG Chemical, ICR18650S3, 2200 mAh)
- Dimension: Handpiece: 255 x 115 x 41.5 cm
Charger: 155 x 82 x 125 cm
- Weight: Handpiece: 136 g
Charger: 144 g
- Operating voltage: 5 V, 1,6 A
AC / DC Adpator
 - Model: UE130415HKKK1-P
 - Input: 100 - 240 V ~, 50 - 60 Hz, 400 mA
 - Manufacturer: Fuhua Electronic

8.2 Environment

Operating Condition

- Temperature: 10~ 40 °C, Relative Humidity: 30~ 80 %, Ambient pressure: 80 ~ 106 KPA

Transportation and Storage Conditions

- Temperature: -10~ 55 °C, Relative Humidity: 30~ 90 %, Ambient pressure: 70 ~ 106 KPA

9. Warranty & Service

9.1 Warranty Information

- ❶ This item is free from manufacturing defects in material and workmanship, provided the product is used in normal conditions and is installed and used in strict accordance with the instructions contained in the Product's Owner's Manual.
- ❷ Warranty Exclusions
 - This warranty does not apply to defects or damages arising by use of the product in other than normal including negligent use, misuse or abuse of the product
 - By fire, earthquake, flood or other Acts of God.
 - By any excessive shocks.
 - By not allowed disassemble, modify.

9.2 Service

In the event that you should have difficulty that requires factory service, please call your dealer or Kulzer GmbH

Division Technique

Philipp-Reis-Straße 8/13

61273 Wehrheim

Tel. +49 (0) 6181/9689-2571 or 2574

Fax. +49 (0) 6181/9689-2964

technik.wehrheim@kulzer-dental.com

www.kerasys.de

Symbole

	Direct current		See instruction for use
	Attention, consult accompanying documents		Manufacturer
	Fragile - handle with care		General symbol for recovery/recyclable
	Manufacturer date		CE mark
	Serial No.		Type B applied part